

# CLEANASCOPE® Advantage

Transport & Short-term Storage System | **Dry & Store**





## THE COMPLETE CIRCLE OF PROTECTION

As the global vanguard in infection prevention, only Cantel Medical delivers the Complete Circle of Protection, a full-value, proactive partnership dedicated to helping you remove risk, streamline operational efficiencies and optimize your success.

## Clear Identification of Clean Versus Contaminated Endoscopes




### Confidence is Knowing it's Clean at a Glance

The CLEANASCOPE® Advantage Transport and Short-term Storage System protects your endoscopes and other delicate equipment by minimizing direct handling and reducing contact with airborne contaminants.

Its patented design creates a protective environment for each disinfected endoscope. Green (clean) and red (contaminated) color-coded liners immediately identify clean or contaminated equipment.

## Optimized Endoscope Workflow Maximizes Infection Prevention

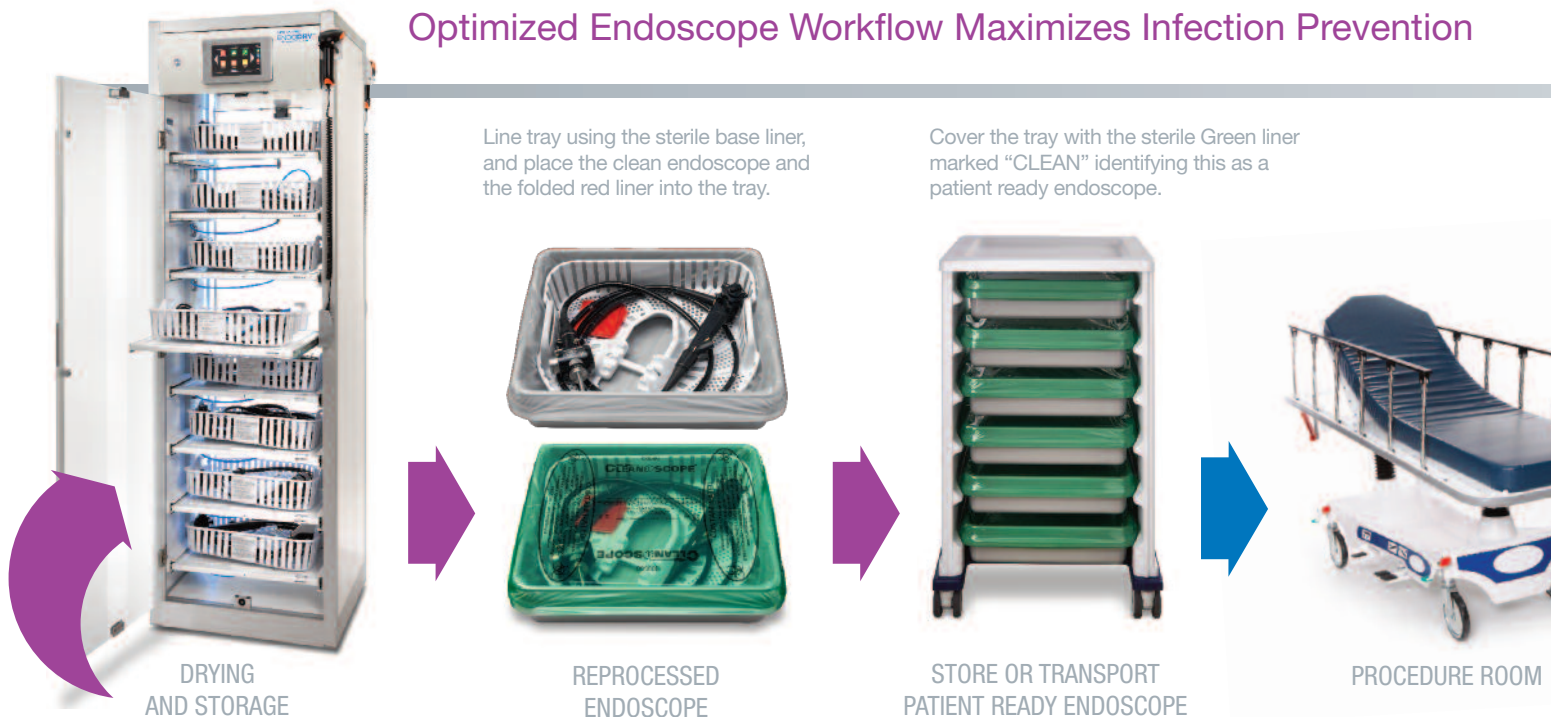



## DRY & STORE

Bacteria pose significant risk to endoscopes during transport and storage. Cantel's transport, drying and storage solutions are designed to protect valuable inventory, reduce cross contamination touchpoints, eliminate moisture in the endoscope channels and control humidity. Humidity or moisture in endoscope channels is known to aid bacterial growth.

# CLEANASCOPE® Advantage

Transport & Short-term Storage System | **Dry & Store**

Trays fit perfectly into the contoured cart top

Provides protected short-term storage of clean scopes and equipment

Hard lids available for added protection when transporting outside department

Available in 5, 6 and 10 tray options



Robust reusable tray easily fits large endoscopes and the ADVANTAGE PLUS® AER cassette



Three-part, single-use sterile packed liner system reduces the potential for cross-contamination

Large locking caster, wheels make transport easy and smooth


Post procedure, flush & wipe the endoscope using INTERCEPT® Detergent and place dirty endoscope inside tray. Cover tray using the Red liner for clear identification of "CONTAMINATED" endoscope.



CONTAMINATED TRANSPORT



MANUAL CLEANING

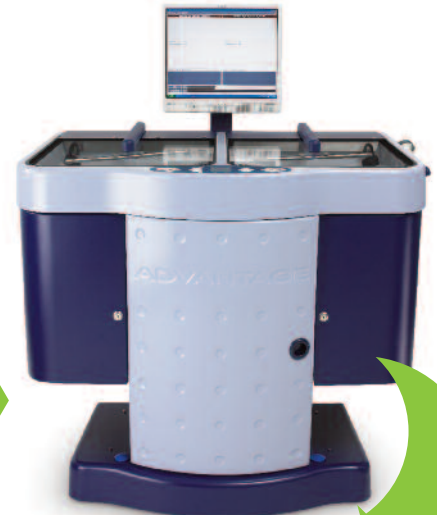

FIT SECURELY IN ADVANTAGE REPROCESSOR



### CLEANASCOPE® Cart Ordering Information

Part Number	Product Description	Dimensions
103470	Five tray cart (trays included)	34.7H x 25.2W x 22.1 D (in)
103466	Six tray cart (trays included)	40.5H x 25.2W x 22.1 D (in)
103467	Ten tray cart (trays included)	63.5H x 25.2W x 22.1 D (in)



### CLEANASCOPE® Liners & Accessories Ordering Information

Part Number	Product Description	Units Per Box
103585	Sterile Liner Set	200 Sterile liners/case
103604	Dispenser, (wall mounted)	1
103456	Tray	1
103457	Tray Lid, clear	1
CAS-1000	Advantage Cassette	1

*“Transport the soiled endoscope to the reprocessing area in a closed container that prevents exposure to staff, patients or the environment to potentially infectious organisms”*

ASGE Quality Assurance in Endoscopy Committee, 2011 and SGNA Standards of Infection Control in Reprocessing of Flexible Gastrointestinal Endoscopes 2012

*“The sterile liner system is one of the most important reasons why many facilities are adopting the CLEANASCOPE® System – the color coded liners take the guesswork out of endoscope reprocessing”*

Libby Thomson, former Lead Nurse, Bowel Cancer Screening Centre and Nurse Endoscopist

ADVANTAGE PLUS® and INTERCEPT® are registered trademarks of Medivators Inc.  
CLEANASCOPE® is a registered trademark of Cantel (UK) Limited.

**CANTEL**  
Global Headquarters  
150 Clove Road - 9th Floor  
Little Falls, NJ 07424-2139  
Phone: +1.973.890-7220

**MEDIVATORS**  
Global Headquarters  
14605 28th Avenue North  
Minneapolis, MN 55447-4822  
Phone: +1.763.553.3300 Fax: +1.763.553.3387

**TO PLACE AN ORDER**

Call: 1.800.328.3340 (Customer Service)  
Fax: 1.800.686.8493  
Email: [custserv@medivators.com](mailto:custserv@medivators.com)